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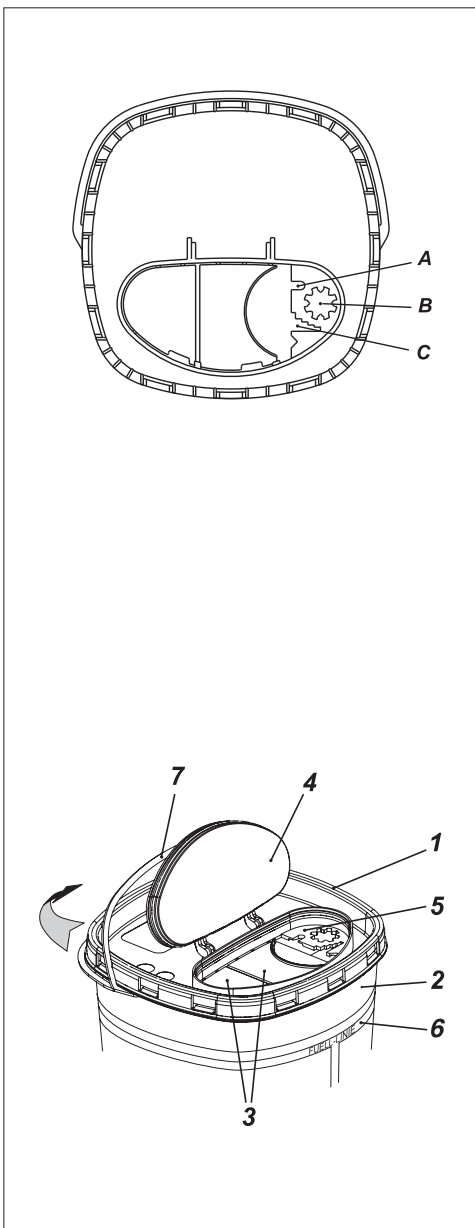
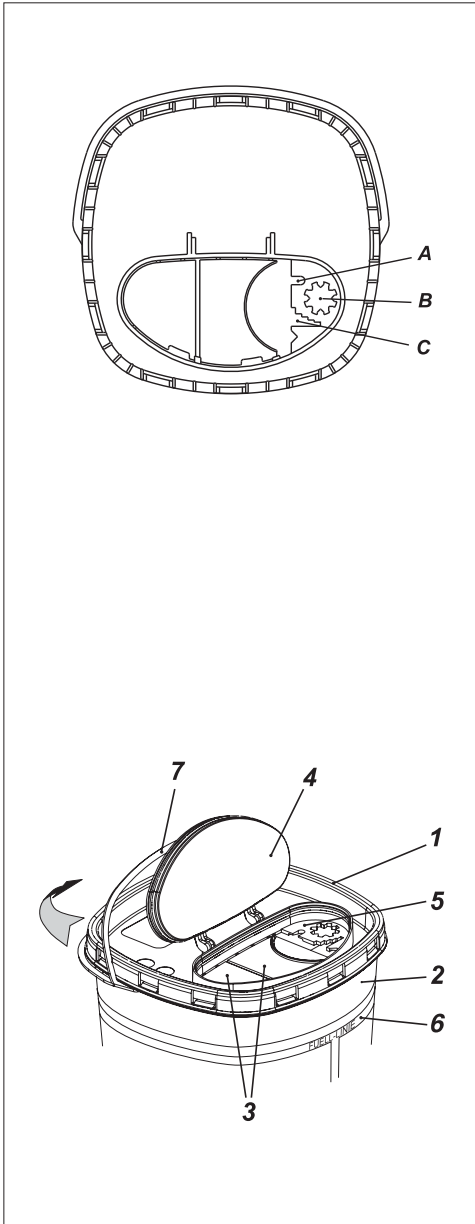
# EN Instructions for Use: Multi-Safe *twin*

## 1. Intended purpose and normal use

The disposal box is exclusively intended for the disposal of pointed and sharp items like, for example, needles and scalpel blades for medical use.

## 2. Safety instructions

- Use only boxes adequately sized to accommodate the items to be disposed.
- Keep the box beyond the reach of children.
- Ensure that the lid (1) is assembled and locked into place before filling.
- Attach the box to recommended base adapter or wall holder, or place into the Safety-Tray to protect it against overturning.
- Ensure that items are placed into the box in a vertical direction only.
- Scalpels must be placed into the box with utmost care. Using too much force or filling other objects on top involves the risk of jamming and damage to the box walls or base.
- Do not use force to press objects into the box.
- Do not fill with liquids.
- Do not fill the box above the maximum filling line (5) (health hazard!).
- Do not touch the inside of the box with your hands or otherwise (health hazard!).
- Do not throw.  
Do not shake.  
Do not drop.
- Ensure that no objects are jutting out before closing.
- Do not press the box against your body (e.g. chest) for final closure.
- Do not press the box together from the outside when full.
- Use the handle.
- Prior to disposal of the box, ensure that lids 1 and 4 are securely closed

## 3. Assembly (lid assembly)

Prior to filling, press the lid (1) firmly onto the box base (2) until it audibly and visibly locks into place along its entire circumference. The size of the filling opening can be customized by breaking the two perforated sections (3) prior to lid assembly.

## 4. Daily use

For daily use, lightly press the day lid (4) into the filling opening (5) to enable re-opening.

## 5. Final closure of day lid (4)

After filling [Warning: Do not fill above the maximum filling line (6)!], proceed as follows: Place the box on a solid surface and press the day lid (4) firmly into the filling opening (5) until the day lid audibly locks into place along the entire circumference of the box.

## 6. Fill line (6)

The upper section of the box features a fill line (6). For safety reasons, do not fill the box above the maximum filling level (6).

## 7.1 Holding fixtures

A base adapter, wall holder or Safety-Tray is available to protect the box against unintentional overturning (Refer to separate User Guide: "Holding Fixtures").

## 7.2 Carrier handle (7)

The carrier handle (7) incorporated in the upper box section can be released by slightly pulling upwards and ensures convenient transport to any disposal site.

## 8. Removal aids

### 8.1 U-shaped recess (A)

for sharps removal from Luer lock systems.

### 8.2 Star-shaped, notched opening (B)

for the disassembly of insulin pens.

### 8.3 Specially shaped recess (C)

for the removal of needles.

## 9. Disposal and identification

Disposal boxes are made from ecologically friendly polypropylene. Incineration does not cause emission of any harmful substances. Observe statutory provisions pertaining to disposal!

## 10. DIN EN ISO 23907

If you use the disposal box in agreement with this directive, you will have to insert the absorbent liner below:

Order number	Packaging pcs./case
78.669	1,000