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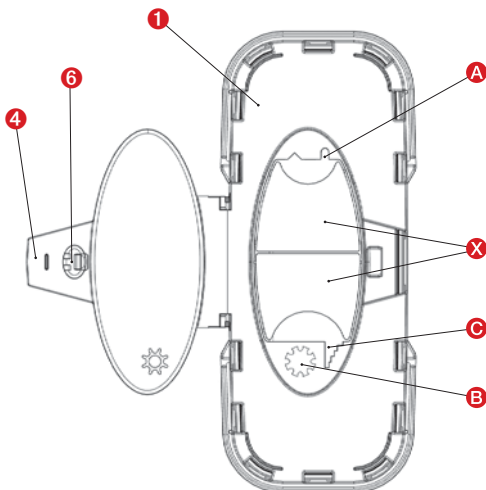
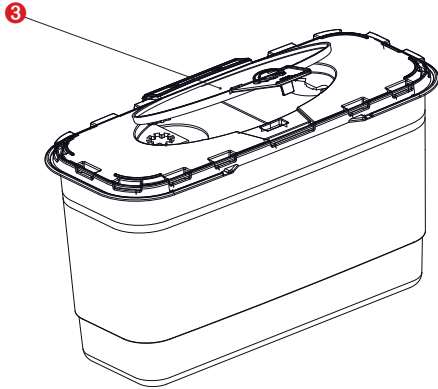
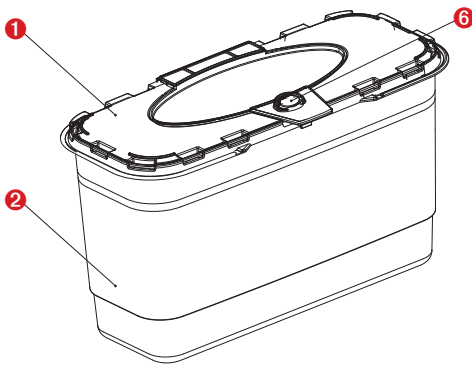
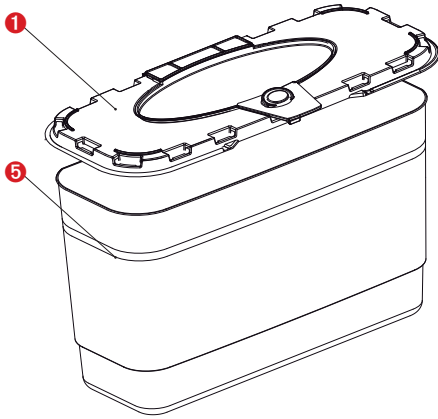
1. Intended purpose and normal use

The disposal box is exclusively intended for the disposal of pointed and sharp items like, for example, needles and scalpel blades for medical use.

2. Safety instructions

- Use only boxes adequately sized to accommodate the items to be disposed.
- Keep the box beyond the reach of children.
- Ensure that the lid (1) is assembled and locked into place before filling.
- Attach the box to recommended wall holder, or place into the Safety-Tray to protect it against overturning.
- Ensure that items are placed into the box in a vertical direction only.
- Scalpels must be placed into the box with utmost care. Using too much force or filling other objects on top involves the risk of jamming and damage to the box walls or base.
- Do not use force to press objects into the box.
- Do not fill with liquids.
- Do not fill the box above the maximum filling line (5) (health hazard!).
- Do not touch the inside of the box with your hands or otherwise (health hazard!).
- Do not throw.
- Do not shake.
- Do not drop.
- Ensure that no objects are jutting out before closing.
- Do not press the box against your body (e.g. chest) for final closure.
- Do not press the box together from the outside when full.
- Use the handle.
- Prior to disposal of the box, ensure that lids (1) and (3) are securely closed.

3. Assembly (lid assembly)

Prior to filling, place the box on a solid surface and press the lid (1) firmly onto the box base (2). The lid (1) audibly locks into place along its entire circumference. The size of the filling opening can be customized by breaking the two perforated sections (X) prior to lid assembly.

4. Daily use

For daily use, press the day lid (3) into the filling opening until its audibly clicks into place. By pushing up the flap (4), a reopening is possible.

5. Final closure of day lid (3)

After filling [Warning: Do not fill above the maximum fill line (5)!], proceed as follows: Place the box on a solid surface, press the day lid (3) into the filling opening until its audibly clicks into place and press the little Button (6) "Press for final closure" (The box is finally closed).

6. Fill line - maximum filling level (5)

The upper section of the label features a fill line (5). Ensure that the box is not filled above this line (5). For safety reasons, do not fill the box above the maximum filling level (5).

7. Holding fixtures

A wall holder or Safety-Tray is available to protect the box against unintentional overturning (Refer to separate User Guide 165: "Holding fixtures").

8. Removal aids

8.1 U-shaped recess (A)

For sharps removal from Luer lock systems.

8.2 Star-shaped, notched opening (B)

For the disassembly of insulin pens.

8.3 Specially shaped recess (C)

For the removal of needles.

9. Disposal and identification





Disposal boxes are made from ecologically friendly polypropylene. Incineration does not cause emission of any harmful substances. Observe statutory provisions pertaining to disposal! Technical modifications reserved.

10. DIN EN ISO 23907

If you use the disposal box in agreement with this directive, you will have to insert the absorbent liner below:

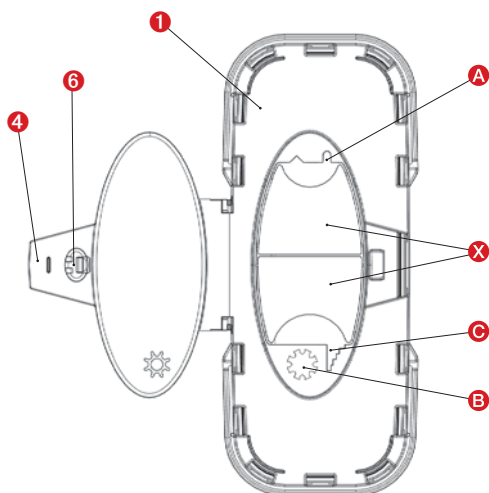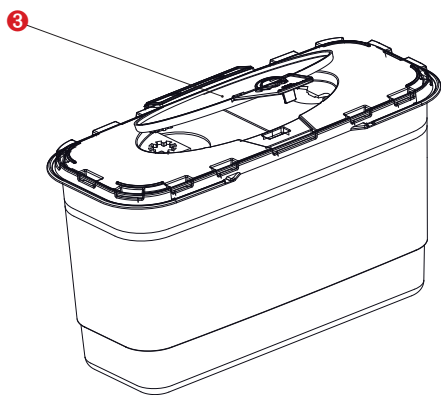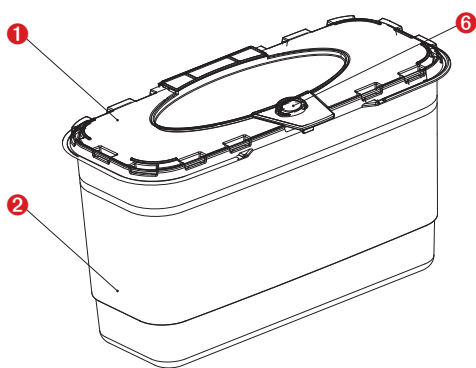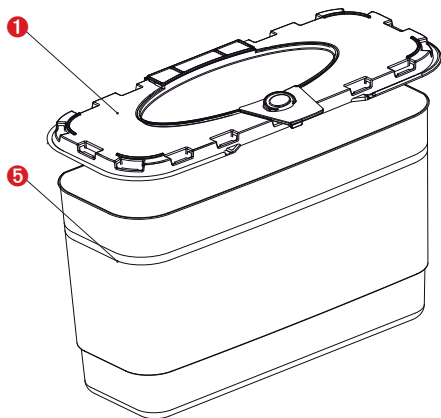Order No.	Packaging
78.669	1,000 pcs.

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